

Positioning Checklist for the Mobile Stander

NOTE: Describe all positioning as if you are standing behind the individual

Use this Positioning Checklist as a convenient way to ensure optimal use of the Mobile Stander. Write notes to customize your instructions for each individual.

INDIVIDUAL'S NAME _____

STAFF NAME _____

DATE _____

THIS POSITIONING CHECKLIST IS INTENDED FOR CONSISTENT USE OF ADAPTIVE EQUIPMENT FOLLOWING ASSESSMENT BY A HEALTH PROFESSIONAL. THE CHECKLIST DOES NOT REPLACE AN INITIAL CLINICAL EVALUATION FOR CORRECT SIZING, COMPONENT SELECTION, AND APPROPRIATE THERAPEUTIC POSITIONING FOR THE INDIVIDUAL. SOME COMPONENT SIZING IS DEPENDENT ON THE BASE MODEL SIZE.

Size

- mini
- small
- medium
- large

Large Wheels

- on
- off

Body Support

Used as anterior support posterior support

Height

Distance from center of lower oval knob to top of colored crossbar (A):


Width

Top: upper lobed knobs (B) adjusted to increase upper body support width

- yes no

Bottom: lower lobed knobs (C) adjusted to increase lower body support width

- yes no

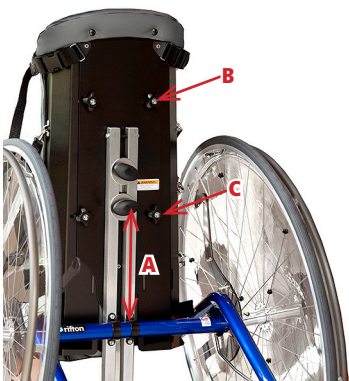

Fleece Strap



- Strap shortened and fleece positioned at upper trunk
- Strap lengthened and fleece positioned behind neck (N/A if in posterior position)
- Fleece not used

Seat Pad



Size small large

Position of attachment straps

Select ring hook position 1–4 (1 is at the top and 4 is at the bottom of the body support.)

Top left strap _____

loose snug

Top right strap _____

loose snug

Bottom left strap _____

loose snug

Bottom right strap _____

loose snug

Additional Seat Pad



Size small large

Position of attachment straps

Select ring hook position 1–4 (1 is at the top and 4 is at the bottom of the body support.)

Top left strap _____

loose snug

Top right strap _____

loose snug

Bottom left strap _____

loose snug

Bottom right strap _____

loose snug

Body Support Center strap



- Buckled to the seat pad (threaded between user's legs)
- Not used

Leg Straps (on body support) Snug Loose Not used

Ankle Straps (on footplate) Snug Loose Not used

Sandals



Left

Distance between sandal toe and side of footplate: _____

Distance between sandal heel and side of footplate: _____

Distance between sandal heel and back edge of footplate: _____

Wedge

- Thick edge at heel
- Thick edge at toe
- Stacked for level lift

Right

Distance between sandal toe and side of footplate: _____

Distance between sandal heel and side of footplate: _____

Distance between sandal heel and back edge of footplate: _____

Wedge

- Thick edge at heel
 - Thick edge at toe
 - Stacked for level lift
-

Arm Prompts (require mounting bracket)

Size small large



Left

Clamp position (L-bar holder)

toward body support away from body support

Distance of clamp from end of mounting bracket: _____

L-bar post position

points away from body support points right points left

L-bar height Specify number of notches visible above clamp: _____

Orientation of arm prompt horizontal angled up angled down

Rotation position (specify as o'clock) _____

Hook and loop strap

across forearm behind elbow no strap

Right

Clamp position (L-bar holder)

toward body support away from body support

Distance of clamp from end of mounting bracket: _____

L-bar post position

points away from body support points right points left

L-bar height Specify number of notches visible above clamp: _____

Orientation of arm prompt horizontal angled up angled down

Rotation position (specify as o'clock) _____

Hook and loop strap

across forearm behind elbow no strap

..... **OR**

Hand Loops (require mounting bracket)



Left

Orientation of hand loop vertical tilted forward tilted backward

Clamp position (hand loop holder)

toward body support away from body support

Distance of clamp from end of mounting bracket: _____

Hand loop height Specify number of notches visible above clamp: _____

Right

Orientation of hand loop vertical tilted forward tilted backward

Clamp position (hand loop holder)

toward body support away from body support

Distance of clamp from end of mounting bracket: _____

Hand loop height Specify number of notches visible above clamp: _____

Communication Tray (requires mounting bracket)



Clamp position

to user's left

facing toward body support

facing away from body support

to user's right

facing toward body support

facing away from body support

Centered (can only face away from body support)

Post height

Specify number of notches visible above clamp: _____