



**Rifton Wave** (Z220 with tub transfer base Z228)

NEW

# The Rifton Wave

## A Bathing & Transfer System

**Hygiene and health go together.** When we make bathing easier, our clients are healthier. The all new Wave system has been thoroughly reimagined to improve bathing and showering – for the client and for the caregiver.

Responding to requests from therapists and caregivers, we widened these new bath chairs to accommodate larger clients. And to protect skin integrity we added generous padding under soft knit fabric.

As well as mounting the bath chair on a tub stand or rolling shower stand we've added an ingenious new tub transfer base to completely change the way you transfer and bathe large clients. There's nothing like it.



Find letters of medical necessity and informative articles at:

[www.rifton.com/wave](http://www.rifton.com/wave)

“ The Wave Bath Chair is terrific! In the past, lifting a child into a tub has been a back killer, but the new transfer base eliminates this hazard entirely. The casters allowing lateral movement in the tub are a welcome addition, and the fact that I can easily collapse the base and hang it on the wall is a fabulous space saver. ”

Linda K. Miller, DPT  
Queensbury, New York



### Unique Features & Benefits

Include these in your letters of medical necessity:

- **Extra wide seat** with open sides, 15" width for small and medium, 17½" for the large.
- **Large size accommodates adults** up to 6'2" and 250 lbs.
- **Completely configurable;** all sizes of the Rifton Wave can fit on any of the 3 base options.
- **Optional calf rest** adjusts independently of seat angle.
- **Tub transfer base** enables users – after being placed in the bath chair using a mechanical lift – to be rolled back and then rotated over the tub for showering.
- **Conversion kit** allows user to economically move up from small to medium size by just purchasing a new back, seat and cover.
- **Least costly alternative** to remodeling a bath room or assisted living, without resorting to bed baths.

# Benefits and features of the Wave

## No tools required

*Who wants assorted tools cluttering the bathroom? The Wave is truly tool free, not only for initial assembly but also for all adjustments while in use.*



### Generously sized

Wide seat with open design enables larger users to fit comfortably. New seat width on large size is 17½".



### One-hand adjustment

Easily adjust the backrest angle with one hand. Allows caregiver to keep the other hand on the user for safety and reassurance.



### Full leg support

Provide full support for the lower legs with the optional calf rest. Adjust it independently of the seat by pushing the gray buttons and rotating up or down. The calf rest must be chosen at the time of the original purchase.



### Typical bath tub height range

- 18 in.
- 14 in.







**Tub stand**



**Shower stand**



**Tub transfer base available Summer 2017**



**Infinite adjustability**

Ideally position your client using the separate back angle, seat angle, and calf rest angle adjustments. Change the seat angle and height by rotating the front and/or back legs.



**More seat heights**

For an intermediate seat height, put rear legs up flat and front facing legs down. To get the chair low in the tub, put both front and back legs up. Fold the chair completely flat for storage.



# Required components

## Bath chair size and color



## Choice of chest strap



### Without lateral positioning

The simple chest strap safely positions the user in the Wave bath chair. It attaches with hook and loop where needed along the full height of the back, and buckles securely behind.



### With lateral positioning

Rifton designed this chest strap with adjustable lateral straps to give additional lateral support to help users maintain better trunk alignment in the bath chair. The soft fabric laterals will not get in the way during transfers.



## Accessories



### Head blocks

Optional head blocks provide lateral support and positioning for the user's head.



### Leg straps

Use the fully adjustable leg straps to position your client's hips in abduction (or just to provide additional security).



### Calf rest

The new calf rest option provides more support for the lower legs and is especially needed when the bath chair is used on one of the stands.

**Note:** Calf rest must be chosen at the time of the original purchase. It cannot be added later.



### Tub stand

The optional tub stand raises the bath chair either 5½" or 11½" higher in the tub depending on the position of the tub stand legs.



### Shower stand

The shower stand allows you to roll your client into the shower stall at a height that allows better cleaning. Better cleaning means better health. Four-inch locking casters provide smooth, easy maneuverability.



## Accessories (continued)

### Tub transfer base available in Summer 2017

**Bathing equipment for people with disabilities tends to be big and bulky**, taking up far too much space in bathrooms that are shared by everyone in the family. Often this means that families are forced to take shortcuts or skimp on hygiene care for their loved ones with special needs. And poor hygiene often results in poor health.

Now with Rifton's ingenious new tub transfer base you can perform healthy transfers—either manually or with a mechanical lift—without crowding your bathroom with permanently mounted equipment. When bath time is over the base collapses and hangs compactly on the wall.



#### Unique two-stage movement



The tub transfer base starts in the position farthest out of the tub. It is then rolled back over the tub, and the brakes are set.



As the upper portion with the bath chair is rotated 90°, it moves approximately 1' back over the tub in one motion.



The tub transfer base will fit most bath tubs up to 19" high by adjusting the leg length to clear the edge of the tub.





**1. With the client secured in the chair,** push the transfer base to the back of the tub as far as it will go and lock the wheels.



**2. Press either of the gray pivot latches on the transfer base** and start to rotate the bath chair and client over the tub.



**3. As the bath chair rotates,** it continues to move back over the tub. Gently lift the client's legs to clear the edge of the tub.



**4. In the final position** the bath chair is over the tub, and can be adjusted as desired before showering.

## Free up the tub.


**Rifton's folding,** storable tub transfer base makes it possible for everyone in the family to still use the tub.

When you are finished just remove the base from the tub, fold the legs, and hang it on the Rifton wall hook provided.





# Dimensions of the Wave

User dimensions (inches)	Z210 small	Z220 medium	Z230 large
Height	36–46	40–56	50–74
 <b>Key user dimension: height</b> Select the appropriate bath chair by the user's overall height. Choose the size that allows for growth. <b>Important:</b> Make sure that seat depth, width and back height are adequate. The user's weight must not exceed the maximum working load.			
	Item dimensions (inches)	Z210 small	Z220 medium
Seat depth	10½	13	17
Seat width	15	15	17½
Back height	23	26½	32
Seat height	2–7	2–7	2–7
Seat height (incl. tub stand)	12½–18½	12½–18½	12½–18½
Seat height (incl. shower stand)	27½	27½	27½
Seat height (incl. transfer base)	21½–27½	21½–27½	21½–27½
Length of mesh area when flat (without calf rest)	35	41	50½
Length of mesh area when flat (with calf rest)	42½	49	61
Width of base	18	18	21
Length of base when flat	29½	29½	29½
Tub stand height (legs flat)	8	8	8
Tub stand length (legs flat)	39	39	39
Tub stand width (legs flat)	17½	17½	17½
Shower stand height	22	22	22
Shower stand width	27	27	27
Shower stand length	34	34	34
Maximum tub edge height for transfer base	19	19	19
Maximum working load (lbs)	75	150	250
Back and seat angle adjustments		0°, 15°, 30°, 45°, 60°, 75°	



**Without calf rest**  
(Z269)

**With calf rest**  
(Z267)

**Conversion kits** are the smart solution for growing children. For a fraction of the cost you can convert a small to a medium bath chair.

We'll send you a new larger fabric cover, back and seat frames and chest strap with instructions on how to remove the old and install the new parts on your existing base.



The Wave has a washable fabric cover that is easily removed for cleaning.



All three sizes are available in two beautiful colors, blue and pink.

# How to buy

- 1. Measure** your client to determine the size you need.  
(See dimension chart for help.)
- 2. Select** the color, appropriate bath chair and chest strap.
- 3. Choose** the accessories you need.
- 4. Add** up the total cost.
- 5. Order** online in minutes at **Rifton.com** or see the back of this catalog for other ways to order.
- 6. Call** us any time at **800.571.8198** if you need assistance.  
We're always here to help.

The tool-free **conversion kit** converts your small chair to medium as your child grows.

**Fabric with calcrest**

Blue	Z266	\$375
Pink	Z267	\$375

(Includes seat, back, cover and chest strap)

**Fabric without calcrest**

Blue	Z268	\$300
Pink	Z269	\$300

(Includes seat, back, cover and chest strap)



		Z210 small	Z220 medium	Z230 large	
<b>Required components</b>	<b>Chair</b>	Z211 \$430 <input type="checkbox"/>	Z221 \$450 <input type="checkbox"/>	Z231 \$520 <input type="checkbox"/>	
	<b>Color</b>	<b>CHOOSE</b> Blue	BLUE <input type="checkbox"/>	BLUE <input type="checkbox"/>	BLUE <input type="checkbox"/>
		<b>CHOOSE</b> Pink	PINK <input type="checkbox"/>	PINK <input type="checkbox"/>	PINK <input type="checkbox"/>
	<b>Chest strap</b>	<b>CHOOSE</b> With lateral positioning	Z212 \$60 <input type="checkbox"/>	Z222 \$70 <input type="checkbox"/>	Z232 \$80 <input type="checkbox"/>
<b>CHOOSE</b> Without lateral positioning		Z213 \$40 <input type="checkbox"/>	Z223 \$50 <input type="checkbox"/>	Z233 \$60 <input type="checkbox"/>	
<b>Optional accessories</b>	<b>Head blocks</b>	Z214 \$85 <input type="checkbox"/>	Z214 \$85 <input type="checkbox"/>	Z234 \$85 <input type="checkbox"/>	
	<b>Leg straps</b>	Z215 \$40 <input type="checkbox"/>	Z225 \$50 <input type="checkbox"/>	Z235 \$60 <input type="checkbox"/>	
	<b>Calcrest</b> (Calf rest must be chosen at the time of the original purchase. It cannot be added later.)	Z216 \$110 <input type="checkbox"/>	Z226 \$125 <input type="checkbox"/>	Z236 \$140 <input type="checkbox"/>	
	<b>Tub stand</b>	Z217 \$250 <input type="checkbox"/>	Z217 \$250 <input type="checkbox"/>	Z217 \$250 <input type="checkbox"/>	
	<b>Shower stand</b>	Z218 \$680 <input type="checkbox"/>	Z218 \$680 <input type="checkbox"/>	Z218 \$680 <input type="checkbox"/>	
	<b>Tub transfer base</b>	<b>CHOOSE</b> Pivot to right	Z227 \$850 <input type="checkbox"/>	Z227 \$850 <input type="checkbox"/>	Z227 \$850 <input type="checkbox"/>
		<b>CHOOSE</b> Pivot to left	Z228 \$850 <input type="checkbox"/>	Z228 \$850 <input type="checkbox"/>	Z228 \$850 <input type="checkbox"/>
	<b>Additional head blocks</b>	Z214 \$85 <input type="checkbox"/>	Z214 \$85 <input type="checkbox"/>	Z234 \$85 <input type="checkbox"/>	
	<b>Additional fabric cover</b>	Z219 \$180 <input type="checkbox"/>	Z229 \$190 <input type="checkbox"/>	Z239 \$200 <input type="checkbox"/>	
	<b>Additional chest strap</b>	<b>CHOOSE</b> With lateral positioning	Z212 \$60 <input type="checkbox"/>	Z222 \$70 <input type="checkbox"/>	Z232 \$80 <input type="checkbox"/>
		<b>CHOOSE</b> Without lateral positioning	Z213 \$40 <input type="checkbox"/>	Z223 \$50 <input type="checkbox"/>	Z233 \$60 <input type="checkbox"/>
<b>Total</b>	<b>Z210</b> \$ _____	<b>Z220</b> \$ _____	<b>Z230</b> \$ _____		