



# **Compass Chair**

**It's the perfect inclusion chair.** The rock-solid frame is built to withstand years of demanding use. The contoured seat and sturdy armrests provide reassuring boundaries, giving mildly involved students the support they need to feel secure and comfortable. At the same time, its simple and elegant construction lets it blend easily into any classroom environment. It looks almost like a regular classroom chair — it's designed to.

The five available sizes accommodate everyone from small children to adults, and the adjustable legs allow fine-tuning for optimal positioning.



Find important details and informative facts about the Compass Chair at: www.rifton.com/compasschair



We have had trouble getting one of our students to sit even for a few moments, which, of course, makes learning very difficult. We've tried different seating options but nothing seemed to work. Imagine our surprise when he saw the Compass Chair for the first time and, entirely on his own, went and sat in it until we had to ask him to get up!

Kathleen Marshall Anderson Center for Autism Staatsburg, NY



## **Compass Chair features**



The Compass Chair comes in five sizes.





The low friction feet are made of hard plastic and slip on over the standard rubber feet to allow the chair to slide easier.



Stability feet attach to the front and back legs to accommodate your more active, tip-prone students.

### **Components**



#### **Seat belt**

The optional seat belt clips into the seat when needed for safety or postural support.





#### **Stability feet**

The stability feet enlarge the chair's base of support. They are adjustable like the standard chair legs.





#### **Pelvic harness**

An alternative to a typical seat belt, Rifton's unique pelvic harness secures a student's hips and upper thighs without pressure on the abdomen. The pelvic harness provides a secure, comfortable base for correct posture.





#### **Low friction feet**

The low friction feet allow the chair to slide easier over different floor surfaces for easier positioning and reducing the chance of tip-over.





### **Adjustable legs**

The legs provide two inches of height adjustment to ensure optimal seat height and allow for growth. Both chairs in this picture are size 3.



# **Compass Chair dimensions**

User dimensions (inches)		R310 size 1	R320 size 2	R330 size 3	R340 size 4	R350 size 5						
Height		36 – 47	43 – 54	49-60	55-68	62+						
Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth.  Important: Make sure that seat width, depth, and height are adequate for each user,												
and that the user's weight does not exceed the maximum weight recommended.												
Item dimensions (inches)		R310	R320	R330	R340	R350						
Seat height		9,10,11	10½, 11½, 12½	12½,13½,14½	14½, 15½, 16½	16½,17½,18½						
Seat depth		91/2	11	12¾	14¾	161/2						
Seat width		101/2	121/4	14	161/4	181⁄4						
Backrest height		91/2	11	121/2	141⁄4	16						
Armrest height above the seat		5½	6	7	81/4	8¾						
Armrest length		61/2	71/2	81/2	9¾	11						
Max. working load (lbs)		100	120	175	220	250						

## **Order form**

Use dimension chart to select appropriate size.

		R310 size 1	R320 size 2	R330 size 3	R340 size 4	R350 size 5	
*Compass chair		R311	R321	R331	R341	R351	
*Leg type	Standard	R314	R314	R314	R314 🔲	R314	
CHOOSE	Stability (front and rear)	R316	R316	R316	R316 🔲	R316	
ň	Standard with rear stability	R319	R319	R319	R319 🔲	R319	
Pelvic harness		R313	R323	R333	R343	R353	
Low friction feet (Standard legs only)		R318	R318	R318	R318 🔲	R318	

<sup>\*</sup> Required



### The adaptive inclusion chair

Inclusion is everything. Teachers around the world are doing their best to create a welcoming, secure environment for all their students, wherever they are on the spectrum. And for those with autism or mild physical challenges you'll find a perfect solution in our Compass Chair. With gentle curves providing a sensory boundary, optional pelvic harness, easily adjustable leg lengths, and five seat sizes, now you can offer your students seating that accommodates their individual needs, removing barriers to learning and helping you promote your education goals.



Rifton Equipment PO Box 260 Rifton NY 12471–0260

WWW.RIFTON.COM • 800.571.8198