

Corner Floor Sitter

E610 & E620 Product Manual



CE

 rifton.

WARNINGS

- Correct use of this product requires the prior approval and ongoing guidance of a qualified therapist or physician. Adult supervision is required at all times.
- Straps and supports are provided for the safety of the user. Their use must be closely supervised. The straps and supports need to be carefully adjusted for the comfort and security of the user and can never take the place of the caregiver.
- Restraints—Some people use straps, trays or supports to restrict a child’s movement. This is a behavioral restraint and may raise ethical and legal issues which you should check with your particular facility. Rifton Equipment is not intended for this use.
- Do not use the Abductor accessory as a restraint.
- To prevent strangulation hazard never use Chest Strap before Hip Strap is secured.

IMPORTANT

- Please save this Product Manual. Additional copies are available at www.rifton.com
- Please refer to the Rifton product catalog for our full warranty, or visit www.rifton.com

To order replacement parts

1. Locate the serial number of the product on the small white label.
2. Have this number available when you call **800.571.8198** for your customer service representative.

Use only replacement parts supplied by Rifton Equipment.

Recommended Use

The Rifton Corner Floor Sitter is a simple positioning device for sitting, enhancing balanced weight bearing, and supporting the shoulders so the arms are free for activity. The Sitter positions the child on the floor so he can participate with other children sitting on the floor at “circle time.”

Assembly

Your Corner Sitter (and Abductor if part of your order) comes in one carton.

If your shipment is incomplete, please call Customer Service.

To assemble your Abductor to the Corner Sitter, raise the latch plate under the front of the seat. Slide Abductor bracket into the slot. Let latch drop into desired bracket hole. The bracket has 5 holes for 4" of adjustment (see figure A).

Adjustments

The Hip Strap has a dual adjust buckle. It allows placement of the buckle for the user’s comfort.

The Chest Strap buckle pad is adjustable sideways for comfort. The Chest Strap also adjusts for height using the snaps on the back of the Sitter

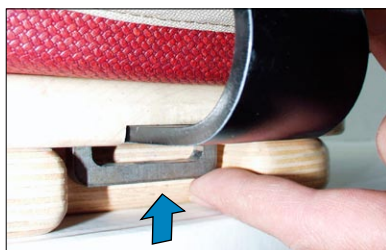


Figure A

Accessory

The E615 Abductor is used with the Rifton Corner Floor Sitter. It provides 2½" of abduction and may help prevent scissoring.



Item Dimensions – Inches (cm)	Small	Large
Overall height	20 (51)	24 (61)
Seat height from floor	2 (5)	2 (5)
Height of back above seat	18 (46)	22 (56)
Seat width	13 (33)	16 (41)
Seat depth	11 (28)	13 (33)
Back to Abductor	4-8 (10-20)	6-10 (15-25)

Maintenance

Periodically inspect for cracks, breaks, loose parts, missing parts, and/or malfunctions. Remove the product from service when any condition develops that might make operation unsafe.

Cleaning

Clean the wood parts by wiping with a damp cloth using a mild detergent or disinfectant. Rinse with a damp cloth and water. Do not use excessive amounts of water. The upholstery can also be cleaned the same way or with any product suitable for imitation leather. Remove straps and clean with a damp cloth or brush using a mild detergent or mild disinfectant. Rinse thoroughly.

Construction

The wooden components are manufactured from hardwood plywood or solid maple and finished with a durable clear finish. All corners and edges are rounded for safety and ease of cleaning. The safety straps are made of sturdy polypropylene webbing, and the locking buckles are made of a tough polymer. Upholstery is urethane foam covered with expanded vinyl. The materials are fire resistant and meet state and federal requirements. The outer fabric is mildew and water resistant.

User Modifications

We recognize that some clients may benefit from modifications made in the field. However, we cannot be responsible for customer-modifications to our products without our supervision, testing, or evaluation.

Rifton Contact Info



Online
www.Rifton.com
Sales@Rifton.com



Phone
800.571.8198
9-5 EST



Mail
Rifton Equipment
PO Box 260
Rifton NY 12471-0260



Fax
800.865.4674