

Corner Floor Sitter

E610 & E620 Product Manual



CE




 rifton®

Contents

Warnings and important information	3
Recommended use	4
Item dimensions	4
Check your order	4
Basic item	5
Hip strap	5
Chest strap	5
Accessory	5
Abductor	5
Maintenance	6
Cleaning	6
Materials	7
User modifications	7

Key for EU users

Use this key to determine which sections of this product manual apply to you.

-  **Technical Users** For professionals who order and set up Rifton products.
-  **Home Users** For care-givers who use Rifton products on a regular basis.
-  **Maintenance Personnel** For anyone who is responsible for service or re-ordering of Rifton products and parts.

WARNINGS

- Thoroughly read and understand the information in this product manual before attempting to use this product. If the procedures and instructions in this product manual are not followed, serious injury could occur.
- A qualified professional must assess the appropriateness and safety of all equipment for each user.
- Correct use of this product requires the prior approval and ongoing guidance of a qualified therapist or physician.
- This product is intended for use by children of immature judgment. Adult supervision is required at all times.
- Do not use this product on rough and uneven terrain, around swimming pools, or near stairways.
- All mobility equipment may become unstable when used improperly.
- Straps and supports are provided for the safety of the user and must be carefully adjusted for comfort and security. The use of straps and supports must be supervised at all times.
- Restraints – Using straps, trays or supports to restrict a child’s movement is considered behavioral restraint, which may raise ethical and legal issues for your facility. Rifton Equipment is not intended for this use.
- Do not use the abductor accessory as a restraint.
- To prevent strangulation hazard never use chest strap before hip strap is secured.

IMPORTANT

- Please save this product manual. Additional copies are available at www.rifton.com
- Please refer to the Rifton product catalog for our full warranty, or visit www.rifton.com

Recommended use

The Rifton Corner Floor Sitter is a simple positioning device for sitting, enhancing balanced weight-bearing, and supporting the shoulders so the arms are free for activity. The Sitter positions the child on the floor so he can participate with other children on the floor at “circle time.”

Item dimensions

Item dimensions – inches (cm)	small	large
Overall height	20 (51)	24 (61)
Seat height from floor	2 (5)	2 (5)
Height of back above seat	18 (46)	22 (56)
Seat width	13 (33)	16 (41)
Seat depth	11 (28)	13 (33)
Back to abductor	4-8 (10-20)	6-10 (15-25)

Check your order

Your Corner Floor Sitter (and abductor if ordered) comes in one carton.

If your shipment is incomplete or in any way damaged on arrival, please call Customer Service, 800.571.8198.

Basic item

Hip strap

The hip strap (A) has a dual adjust buckle. It allows placement of the buckle for the user's comfort (see Figure 5a).

Chest strap

The chest strap buckle pad (B) can be moved sideways for comfort. The chest strap also is height adjustable by using the snaps (C) at the back of the Corner Floor Sitter (see Figure 5b).

Accessory

Abductor

The abductor for the Rifton Corner Floor Sitter provides 2½" of abduction and may help prevent scissoring (see Figure 5c).

Raise the latch plate (D) under the front of the seat. Attach abductor to the Corner Floor Sitter by sliding abductor bracket into the slot. Let latch drop into desired bracket hole. The bracket has 5 holes for 4" of adjustment (see Figure 5d).



Figure 5a



Figure 5b



Figure 5c

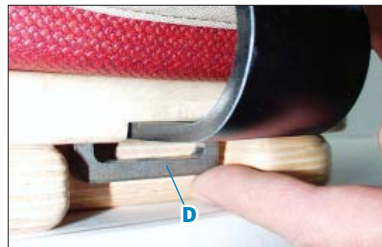


Figure 5d

Maintenance

Periodically inspect for cracks, breaks, loose parts, missing parts, and/or malfunctions. Remove the product from service when any condition develops that might make operation unsafe.

Cleaning

Clean the wood parts by wiping with a damp cloth using a mild detergent or disinfectant. Rinse with a damp cloth and water. Do not use excessive amounts of water. The upholstery can also be cleaned the same way or with any cleaner suitable for imitation leather. Remove straps and clean with a damp cloth or brush using a mild detergent or disinfectant. Rinse thoroughly.

Materials 🧑

- Steel hardware items (nuts, bolts, screws, etc) are typically zinc or nickel plated, or stainless steel.
- Upholstery items (pads, support blocks, padded prompts, etc) are typically fire-retardant polyurethane foam with a fire-retardant cover made from expanded vinyl.
- Frames are typically steel or aluminum tubing, welded together, and coated with a baked-on paint finish. Some frame components may also be stainless steel.
- Tires are tubeless, filled with polyurethane foam, and do not require inflation.
- Straps are typically made of polypropylene or nylon webbing.
- Wooden components are typically birch plywood, solid maple, or laminated hardwood veneers, finished with a clear polyurethane lacquer.
- Tabletops are typically high-pressure laminate (Formica).
- Plastic components are typically injection molded from a variety of industrial resins.

All materials are latex, lead and phthalates free.

User modifications 🧑🏠🧑

We recognize that some clients may benefit from modifications made in the field. However, we cannot be responsible for customer modifications to our products without our supervision, testing, and evaluation.

Rifton Contact Info



Online

www.Rifton.com
Sales@Rifton.com



Phone

800.571.8198
9–5 EST



Mail

Rifton Equipment
PO Box 260
Rifton NY 12471–0260



Fax

800.865.4674

To order replacement parts

1. Locate the serial number of the product on the small white label.
2. Have this number available when you call **800.571.8198** for your customer service representative.

Use only replacement parts supplied by Rifton Equipment.

AG21 ECO 3161 Revision F