

# Attendant Guide Bar

K589 Product Manual





CE

 rifton®

## Key for EU users

Use this key to determine which sections of this Product Manual apply to you.

 **Technical Users** For professionals who order and set up Rifton products.

 **Home Users** For care-givers who use Rifton products on a regular basis.

 **Maintenance Personnel** For anyone who is responsible for service or re-ordering of Rifton products and parts.

## WARNINGS

- Thoroughly read and understand the information in this guide before attempting to use this product. If the procedures and instructions in this guide are not followed, serious injury could occur.
- A qualified professional must assess the appropriateness and safety of all equipment for each user.
- Correct use of this product requires the prior approval and ongoing guidance of a qualified therapist or physician.
- This product is intended for use by children of immature judgment. Adult supervision is required at all times.
- Always remove guide bar when not in use.

## IMPORTANT

- Please save this Product manual. Additional copies are available at [www.rifton.com](http://www.rifton.com)
- Please refer to the Rifton product catalog for our full warranty, or visit [www.rifton.com](http://www.rifton.com)

# Guide bar

## Attaching

1. For maximum stability attach the guide bar centered on the front of the top bar as in Figure 1a.
2. Guide bar is attached by tightening knob (A) to clamp it on.

## Adjustments

See Figures 1a, 1b and 1c.

1. Rotate guide bar until it is behind the child for pushing or in front of child for pulling.
2. Tighten knob (A) to make guide bar stay at the desired height even when you let go of the handle.

**Note:** This product is recommended for use with the mini (K509), small (K501), and medium (K502) Pacer gait trainers.



Figure 1a



Figure 1b



Figure 1c

## User modifications

We recognize that some clients may benefit from modifications made in the field. However, we cannot be responsible for customer modifications made to our products without our explicit supervision, testing or evaluation.

## Rifton Contact Info



### Online

[www.Rifton.com](http://www.Rifton.com)  
[Sales@Rifton.com](mailto:Sales@Rifton.com)



### Phone

**800.571.8198**  
9–5 EST



### Mail

Rifton Equipment  
PO Box 260  
Rifton NY 12471–0260



### Fax

**800.865.4674**

## To order replacement parts

1. Locate the serial number of the product on the small white label.
2. Have this number available when you call **800.571.8198** for your customer service representative.

**Use only replacement parts supplied by Rifton Equipment.**