



# HTS (Hygiene & Toileting System)

**The Rifton HTS** is designed to improve the health of the client and the life of the caregiver.



Scan here to order, request a quote, or see detailed product information.

- Hygienic, simple and affordable
- Promotes ideal positioning for effective toileting
- Use it for toileting, showering, and bathing





# Key features

## 1 Unmatched versatility

The HTS works on, over, or off the toilet—adapting seamlessly to any setting, any toilet style, and a wide range of special needs.



On the toilet

Over the toilet

Off the toilet

The HTS can be securely mounted to any toilet with the appropriate mounting bar, with no tools required. Alternatively, use the mobile base to roll it over a toilet, or add a commode pan for use anywhere outside the bathroom.

## 2 Therapeutic positioning

The HTS provides the safety, security, and support children with special needs rely on, making effective, stress-free toileting possible.



## The HTS in use



Here the HTS is securely mounted on the toilet. Mounting bars allow quick, tool-free attachment of the chair to the toilet base.



The mobile base and commode pan allow the HTS to be used anywhere.

### 3 Convenient hygiene

The open seat pad design simplifies hygiene tasks, making cleaning easier for caregivers while the client remains comfortable and relaxed.



### 4 Room to grow

The HTS accommodates a wide range of sizes, body types, and needs. Adjustable frame height, seat depth, and footrest ensure a comfortable fit as children grow.



The small and medium HTS share the same frame, with seat height adjustable from 15" to 21½".

Seat depth adjusts from 8" on the small HTS to 17½" on the large model.



The natural contours of the chair and cushioned molded foam pads provide both support and comfort, while soft straps protect sensitive skin.



The HTS adjusts to fit a wide range of ages and sizes, providing years of use. Shown here: the same chair adapted for a five-year-old and an eleven-year-old.



# Key features

## 5 More than a toileting chair

The HTS offers comfort, dignity, and caregiver convenience in every toileting and hygiene task.



The small and medium HTS seats fit the tub stand for safe in-tub care, while the tilt-in-space base allows easy showering and hair washing.

## 6 Portable

The portability kit delivers the hygiene and support essential for successful toileting, giving clients and families the freedom to travel with confidence.



**Video:** See how the HTS portability kit sets up over a toilet.



## The HTS in use



The compact footprint of the tub stand fits both narrow and wide tubs.



The portability base includes a lightweight folding frame and carry bag, making toileting possible while traveling.



Scan for sizing  
and pricing.

## 7 Easier lifts and transfers

The flip-up footboard and gas-spring–assisted forward-tilting seat make transitions easier. Swing-away laterals and removable armrests provide added convenience for safe, efficient transfers.



## 8 Hygienic and easy to clean

The HTS features waterproof construction and quick disassembly, allowing every part to be washed and disinfected with a simple 10% bleach solution.



Lifts and transfers present a major challenge to successful toileting. The HTS makes them easier with a variety of features.



The HTS adapts to every stage of growth, offering a custom fit for clients of all sizes and needs.



# HTS options

## Frame size options



**Small Z110**



**Medium Z120**



**Large Z130**

## The HTS in use

---



# HTS components



Scan for sizing  
and pricing.

## Base options



**Mobile**  
(tilt-in-space with footboard)



**Mobile**  
(non-tilt with footboard)




**Stationary**  
(non-tilt with footboard)

**Note:** check clearance if using over a toilet.



**Tub base**  
(non-tilt, for medium and small HTS seats)

Use the **mounting bar** to attach the HTS seat to round-front or elongated toilets. The easy on/off design lets caregivers quickly position or remove the seat.

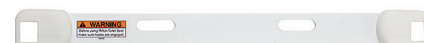

### Mounting bar for round front toilet

For round-front toilets with a distance of 17" or less from the seat bolt centerline to the front edge of the bowl.



### Mounting bar for elongated toilet

(compatible with small and medium HTS) For elongated toilets with a distance greater than 17" from the seat bolt centerline to the front edge of the bowl.

## Components and options



**Rear-closed**



**Rear-open**



**No-hole**



**Soft seat pad**

### Seat pads

Waterproof foam pads remove easily for cleaning. Choose a rear-open pad for hygiene care while seated, or rear-closed and no-hole pads for added support. A rigid hole filler is available for medium and large models. The soft seat pad offers extra cushioning for sensitive skin.



### Armrests

Adjustable armrests are available in three heights and remove easily for transfers.



# More HTS components



## Push handles

Allow caregivers to easily move a client seated in the HTS.



## Headrest

Provides support for limited head control; adjusts in height, depth, and angle.



## Butterfly harness

Offers maximum trunk support, security, and freedom of movement.



## Anterior support/tray

Encourages forward-leaning posture to support toileting.



## Laterals and chest strap

Provide trunk support for clients with limited torso control.



## Hip guides

Reduce seat width, align the torso, and improve trunk control; removable for transfers.



## Large hip guides (thick pads)

Thicker pads allow narrower seat width adjustment.



## Pan

Durable off-toilet pan with pour spout for easy cleaning.



## Bowl adapter

Fills the front gap between the HTS seat and the toilet, ensuring waste drops cleanly into the bowl. It is especially useful on wall-hung and round-front toilets with the large HTS.



Standard

Splash guard

Dynamic

### Deflectors

Choose from standard, splash guard, or dynamic deflector (with abductor) for dependable containment.



### Abductor

Separates knees, supports leg muscles, and promotes healthy hip and joint alignment.



### Calf rest and ankle straps

Enhance comfort, prevent sliding, and provide safe, moderate movement control.



### Footboard for use with mounting bar

Adjusts in height and angle to support the client's feet.



### Large casters

Improve mobility by clearing higher thresholds and obstacles.



### Portability base with carry case

Lightweight folding base and carry bag make toileting possible while traveling.



### Tub base

Tilts the seat back 5° for showering and hair washing; fits narrow and wide tubs.

## Need support?

The Rifton website offers everything you need:

- Evidence-based articles
- Sample letters of medical necessity
- Video tutorials and training resources
- Product instruction guides
- Clinical resources and best practices



Scan here to order, request a quote, or see detailed product information.