


Key Measurements for Rifton Seating Products

Use this diagram to get the measurements

you need to select your Rifton seating product. Seat depth is particularly important; allow for some space behind the knee. Also consider hip width and support for back, shoulders and head. Most Rifton products have a wide range of adjustment with considerable overlap between sizes. If the client is within the range of more than one size, it is usually preferable to choose the larger size to allow for growth.

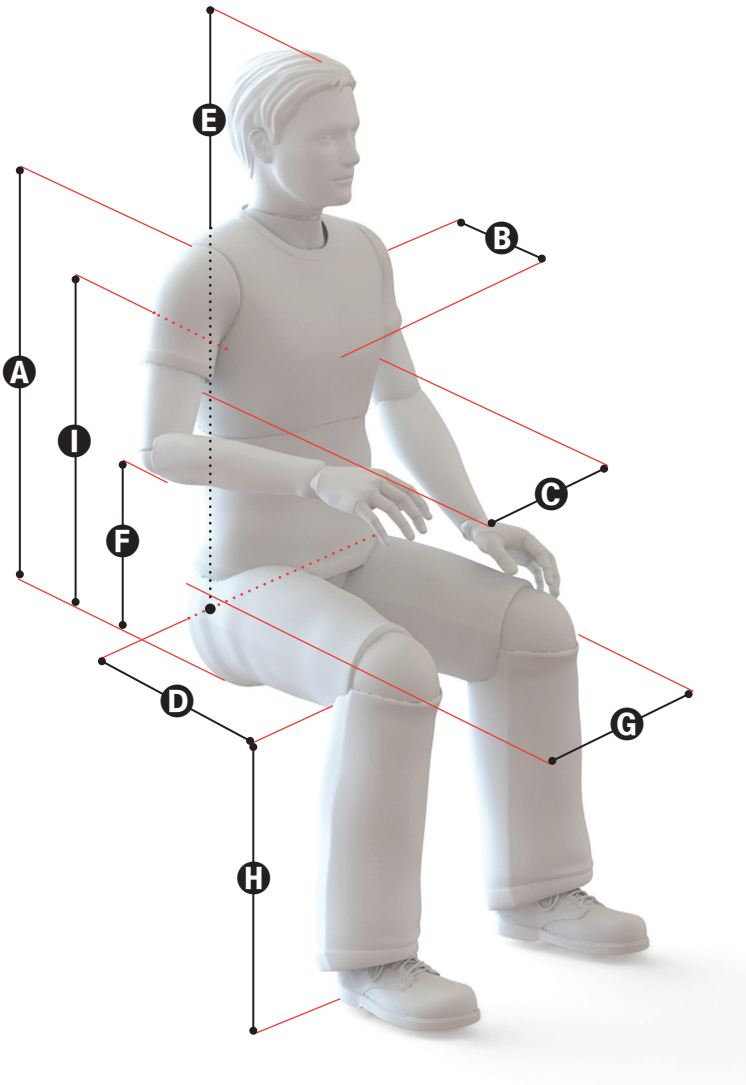

CLIENT'S NAME

DATE

CLIENT'S WEIGHT

HEIGHT

COMPLETED BY

	LEFT	RIGHT	BOTH
A: Seat pan to top of shoulder:			
B: Chest depth:			
C: Chest width:			
D: Seat depth:			
E: Seat pan to top of head:			
F: Seat pan to elbow:			
G: Hip width:			
H: Behind knee to heel:			
I: Seat pan to axilla:			

Comments:



Rifton Compass Chair Measurements

Measurement	Chair Dimension	Size 1 R310	Size 2 R320	Size 3 R330	Size 4 R340	Size 5 R350
A Seat pan to top of shoulder	N/A	N/A	N/A	N/A	N/A	N/A
B Chest depth	N/A	N/A	N/A	N/A	N/A	N/A
C Chest width	Backrest width	9½" (24 cm)	11" (28 cm)	12½" (32 cm)	14½" (37 cm)	16¼" (41.5 cm)
D Seat depth	Seat depth	9½" (24 cm)	11" (28 cm)	12¾" (32.5 cm)	14¾" (37.5 cm)	16½" (42 cm)
E Seat pan to top of head	N/A	N/A	N/A	N/A	N/A	N/A
F Seat pan to elbow	Armrest height above seat	5½" (14 cm)	6" (15 cm)	7" (18 cm)	8¼" (21 cm)	8¾" (22.5 cm)
G Hip width	Seat width	10½" (27 cm)	12¼" (31 cm)	14" (36 cm)	16¼" (41.5 cm)	18¼" (46.5 cm)
H Behind knee to heel	Seat height above floor – in 1" increments	9" – 13" (23 – 33 cm)	10½" – 14½" (27 – 37 cm)	12½" – 16½" (32 – 42 cm)	14½" – 18½" (37 – 47 cm)	16½" – 20½" (42 – 52 cm)
I Seat to Axilla	Backrest height	9½" (24 cm)	11" (28 cm)	12½" (32 cm)	14¼" (36.5 cm)	16" (41 cm)